

ZipperLine LEMO cable with Apex Locator Hub: Apex Locator compatibility

Dear user,

The following list details the various models that were found to be compatible with the Apex Locator Hub of the ZipperLine Cable.

The list also specifies the type of required adapter (male-male or female-male) for each model:

Apex locators		Adapter type compatibility		Comments
Manufacturer	Device	Male - Male	Female - Male	
Dentsply	Ray-plex	x		
Dentsply	Detect	x		
Dentsply	ProMark	x		
Morita	Root ZX Mini	x		
Morita	Root ZX II OTR Module	x		Without OTR module
Kerr Endodontics	Apex ID	x		
Coltene	CanalPro	x		
SybronEndo	Mini Apex Locator		x	Extension only
NSK	iPex	x		
MicroMega	Apex Pointer	x		
Brasseler / NSK	PAL	x		
Osada	Apit / Endex	x		
MedicNRG	Apex NRG	x		
DiaDent	Naviroot		x	Extension only
Ultradent	EndoEZE Find	x		

Notes:

- The adapter type needs to be determined prior to the supply of the apex locator.
- The apex locator function can only be operated with the rotary micromotor-handpiece, when the rotary head is latch-type (to provide proper conductivity).
- The apex location function is not available for the SAF System micromotor-handpiece, and is not required, since the Self-Adjusting Files do not shift the position or shorten the length of the canal, nor do they create the "pulling in" sensation common to many rotary instruments.